What is a

THE SAFE CHOICE FOR YOU

As

THE DIAPHRAGM

PARDHI RESEARCH PVT. LTD.

We congratulate you on your decision to have acquired a diaphragm. Indeed the diaphragm is a time-tested, reliable and safe contraceptive. It respects your sexual feelings and does not interfere with your desire to make love spontaneously. If you wish, no one besides you has to know that you are using it. With the diaphragm, you have got a device which is totally in your control.

This booklet gives complete information about the diaphragm. Go through it thoroughly before you to use the diaphragm and if you have any questions or suggestions, do write to us .

What is a diaphragm?

A diaphragm is a cup shaped device made of silicone with a spring inside the rim. You are probably familiar with silicone since the nipples of feeding bottles are also made of this material.

As you know, the diaphragm is used by placing it inside the vagina so that it covers the mouth of the uterus (the cervix). Once inside, the spring around the rim of the diaphragm opens up so that the front end of the rim lodges behind the pubic bone, while the rear end lodges at the deepest end of the vagina.

As the vaginal space differs in size among women, diaphragms come in different sizes ranging from 50-105 mm diameter. A trained health worker can measure and fit you with a diaphragm that is appropriate for you.

How does the diaphragm work?

The diaphragm prevents pregnancies in two ways. Firstly, it covers the cervix and hence prevents the sperm from entering the uterus. Secondly, it acts as a barrier to cervical mucus and prevents it from entering the vagina. Normally the vaginal secretions are acidic in character and have spermicidal qualities. Mucus from the cervix is alkaline. If it is allowed to enter the vagina, it neutralizes the acidity. Further, fertile mucus from the cervix also provides a pathway for the sperm to travel up and meet the egg. Therefore when you are using the diaphragm, the cervical mucus is contained inside it and conception becomes unlikely.

You may have heard that it is necessary for you to use a spermicide with your diaphragm. However, latest research shows that the diaphragm alone is effective in preventing pregnancy.

What are the advantages of a diaphragm?

The diaphragm is a contraceptive completely within your control. You can use it as per your own needs.

It is far safer than IUDs, Pills or Injectables. It does not irritate your body, or interfere with the functioning of any of your reproductive organs.

It does not harm your ability to bear children in any way.

It has no effect on breastmilk, so you can safely use the diaphragm even when you are breastfeeding your child.

It provides protection against some sexually transmitted diseases and cervical cancer.

Getting used to your diaphragm

Before you start using your diaphragm as a contraceptive, spend a few days getting used to it. When you went for a fitting, the health worker / doctor must have taught you how to wear and remove it. Now, all you need is a little practice.

Over the next week, wear and remove the diaphragm daily, slowly increasing the length of time for which you keep it on, until you are comfortable wearing it for the whole day. While wearing the diaphragm, carry out routine activities like bathing, going to the toilet, exercising, going to work, etc. Before removing it, check that it is still in its place.

If the fitting is correct you will not feel any discomfort or pain. In fact, you will not feel the diaphragm at all.

Once you are comfortable wearing and removing the diaphragm, and keeping it on for several hours, try using it during sexual intercourse.

For the first few times, we recommend that you do not rely solely on the diaphragm as a contraceptive. Along with your diaphragm, continue to use whatever method you were using earlier. This will give you some time to make sure that it does not get dislodged during sexual activity, and that you or your partner do not experience any discomfort.

After intercourse, if you find that the diaphragm seems to have moved up your vagina a bit, do not panic. As long as there is no gap between the vaginal wall and the diaphragm, and the mouth of the uterus is covered, you are safe. Once you are confident that the diaphragm stays in place and does not interfere with your pleasure, you can rely on it as a safe contraceptive.


We recommend that you visit your doctor or health worker before you start using the diaphragm as a contraceptive, to check that you are wearing it correctly. Make sure you also tell them if you have experienced any pain or discomfort during its use.

Can my partner feel the diaphragm?


Usually your partner will not be able to feel the diaphragm. The penis does not come into contact with the rim of the diaphragm at all. All that he will be touching is the soft silicone dome of the diaphragm. This does not cause any irritation or discomfort.

Wearing the diaphragm

You can wear the diaphragm in any of the following positions - standing with one leg raised, squatting or even lying down. Choose the position that is most comfortable for you.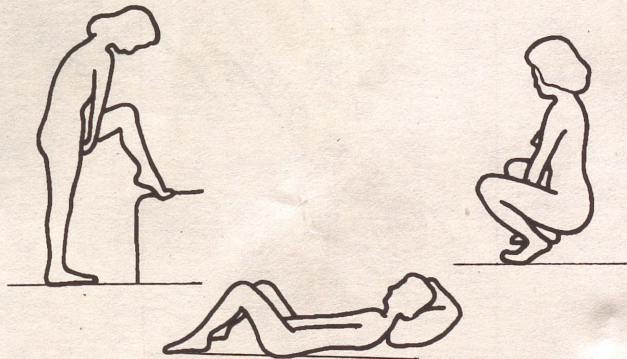


HOW TO WEAR THE DIAPHRAGM

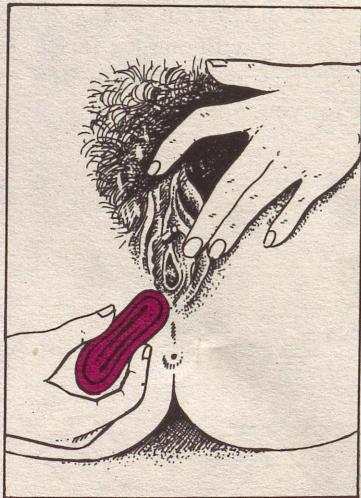
1. Wash your hands and your diaphragm with soap and water.

2. Take the diaphragm in one hand and press the sides of the rim together to make it easier to insert. With your other hand, open the lips of your vagina.

3. Then, slowly push the diaphragm into your vagina. Keep pushing gently downward, along the rear wall of your vagina, until you can feel that the diaphragm has gone all the way in.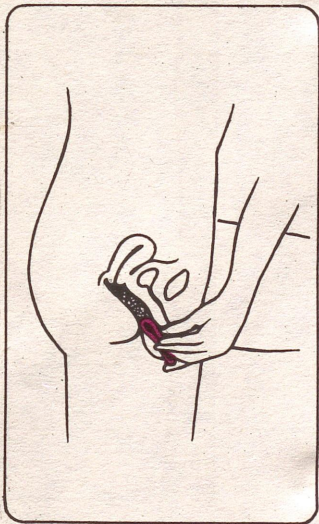

THE DIAPHRAGM INSIDE YOU

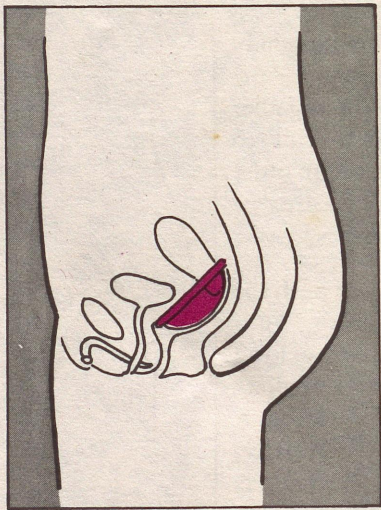
When properly fitted, the diaphragm should cover your cervix. The front of the diaphragm should be in the notch behind your pubic bone, and the rear end should rest against the vaginal wall in the space behind the cervix.

Can the diaphragm get lost inside me?

No, the diaphragm cannot get lost inside you. Your vagina is a closed cavity with only a tiny opening which is the mouth of the uterus. The diaphragm is just too big to pass through this opening. There is no way the diaphragm can travel anywhere inside your body.

Checking the placement of your diaphragm

To check the placement of your diaphragm, insert one finger into your vagina. Check that the outer rim of your diaphragm does not feel loose. It should fit snugly in the notch behind the pubic bone, not on it.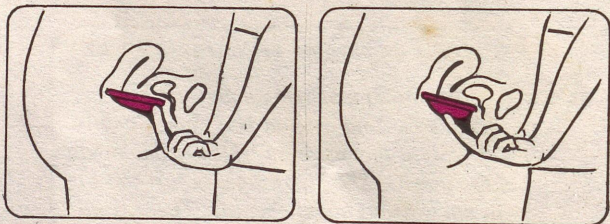

Then take your finger further inside and make sure that your diaphragm is fully covering your cervix.


Removing your diaphragm

To remove your diaphragm, hook one finger over or under the rim, and pull gently. If you have any difficulty in the beginning, do not panic. Relax and try again while bearing down (just as you would do while passing stool).

Routine care of the diaphragm


Each time you remove your diaphragm **wash it with mild toilet soap and clean water**. If you do not wish to use it again at once, let it dry and store it in its box. Do not use strong detergents or powder for cleaning the diaphragm.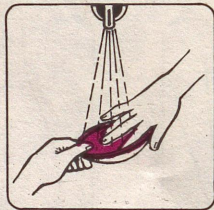

Remember, you should not wear the diaphragm continuously for more than 24 hours. Wash it with soap and water each time before re-use. This is necessary to prevent infections.

Each time you clean your diaphragm check for any cuts and holes, particularly along the rim. Do this by holding the diaphragm against a light or by filling the dome with water and seeing whether any water leaks from it.

Remember, long finger nails can damage a diaphragm.

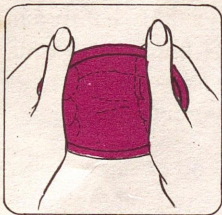A DIAPHRAGM WITH EVEN THE SMALLEST OF CUTS OR HOLES *MUST* BE REPLACED.

Using the diaphragm as a contraceptive

To prevent pregnancy, you must be wearing the diaphragm each time you have intercourse. And keep it on for at least 6 hours afterwards.

The simplest way is to wear the diaphragm everyday, all the time, removing it once a day for washing. Link up this activity with a daily habit to make sure that you remove it once in 24 hours for washing. This will also ensure that your diaphragm is in place whenever there is a chance of having intercourse.

If there is a likelihood of intercourse not taking place eg., your partner is out of town, then you need not wear your diaphragm during these days.

AFTER INTERCOURSE, DO NOT REMOVE THE DIAPHRAGM FOR AT LEAST 6 HOURS.

I forgot to wear the diaphragm yesterday and had intercourse. What should I do?

Having unprotected sex even once can lead to pregnancy. If you forgot to wear your diaphragm, you can wait and see if you are pregnant, and if so, decide whether to have the child or get an abortion. Alternately, you should consult a doctor immediately to be prescribed the "morning after pill", also called "post coital pill".

However, if you find yourself pregnant even after taking these pills, you should seriously consider an abortion. These pills can cause birth defects in your unborn child.

When you should not use the diaphragm

The only times when you should NOT use the diaphragm are ~ When you are menstruating. ~ When you have a cervical or vaginal infection. ~ When you have a urinary tract infection. ~ For 6-12 weeks after childbirth, miscarriage or an abortion because your uterus needs this time to return to its normal size.

Can I use my diaphragm if I have an infection?

If one partner has an infection of the reproductive tract, usually the other gets it too. Therefore, both partners should seek treatment together. It is best to stop wearing the diaphragm and not have intercourse till the infection clears.

If that's not possible, use a condom or as a last resort, a diaphragm to prevent pregnancy.

During Menstruation

We advise you not to wear your diaphragm during menstruation. Such use may lead to a serious though rare condition called Toxic Shock Syndrome (TSS).

Symptoms of TSS are fever above 101 F, diarrhoea, vomiting, rash (like sunburn) and sudden drop in blood pressure which may lead to shock. If you experience such symptoms while using the diaphragm, remove it immediately and seek medical help.

Women who have very short menstrual cycles, i.e. less than 21 days, or very irregular periods, should be aware that they can get pregnant even if they have intercourse during menstruation. If this applies to you, it is best to use some other method of avoiding pregnancy during menstruation such as a condom, practicing withdrawal or having non-penetrative sex.

Side effects and what to do about them

There is only one known side effect of diaphragm use and that is an increased tendency to get a urinary tract infection (UTI). More often than not, this is caused by an ill fitting diaphragm. If your diaphragm is too large for you, the rim could be pressing on the urinary duct, leading to infection. You can recognize a urinary tract infection from the following symptoms:

- * frequent desire to pass urine
- * difficulty in emptying bladder
- * burning sensation or pain while passing urine
- * blood or pus in urine
- * fever

If you suspect that you have a urinary tract infection, get yourself treated by a doctor. Also get the size of your diaphragm checked.

When should you change your diaphragm?

Although the diaphragm lasts for 3 to 5 years, there are certain conditions in which the size you need may change because of a change in the size of your vaginal cavity.

- ~ If you lose or gain 10 lbs (5 kgs) or more of weight.
- ~ After childbirth
- ~ After miscarriage or abortion
- ~ At the beginning of your sexual life. Although it is safe to use the diaphragm at this stage, sexual intercourse tends to enlarge the vaginal cavity. Hence, you will need to be refitted with a new diaphragm of an appropriate size after a few weeks. It may therefore, be more cost effective for you to acquire a diaphragm at this time.

Do I need to sterilise my diaphragm?

It is not necessary to sterilise your diaphragm. It is sufficient to wash it with mild soap and water once a day. Or in the case of use after a long time, just before insertion.

However, if you get an infection in the urinary tract, vagina or any other reproductive organ, you should sterilise the diaphragm after you are cured and before you re-start using the diaphragm.

To do this, boil the diaphragm in clean water for 20 minutes. Make sure the pan has sufficient water to cover the diaphragm well. Drain out the water after boiling and let the diaphragm dry.

***Can I use a lubricant while using
the diaphragm?***

We suggest that you do not use any lubricants with the diaphragm as this could cause the diaphragm to slip out of place. If some lubricant is absolutely needed, use K-Y Jelly or a spermicide.

***Does the diaphragm protect against
diseases?***

The diaphragm does offer protection against some sexually transmitted diseases. Diaphragm users are also less likely to have pelvic inflammatory disease and cervical cancer because it blocks entry to the upper reproductive organs.

When you should seek medical advice

Most women will find the diaphragm safe and trouble-free. But in case you have any of the following problems, please do not hesitate to seek medical advice.

- ~ Difficulty in wearing or removing the diaphragm
- ~ Doubts about the fitting of the diaphragm
- ~ Pain or discomfort during use
- ~ Symptoms of Toxic Shock Syndrome, Urinary Tract Infection or Reproductive Tract Infection.

What if I get pregnant while using the diaphragm?

You may be among a small minority of women who do get pregnant inspite of using the diaphragm regularly. The diaphragm itself will not harm you or your unborn child in any way.

How long will my diaphragm last?

With appropriate care, your diaphragm should last for 3-5 years. All it takes is a little care:

Wash your diaphragm regularly with soap and water. Do not use strong detergents or powder.

Take care not to damage it with long finger nails.

When not in use, store the diaphragm in its box, away from direct sunlight or heat.

Remember, a diaphragm must be changed if it develops holes or cuts, or if the spring on the rim becomes loose.

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